

Product Data Sheet



virion\sersion

Bulk antigen Mycoplasma pneumoniae

Order no.	BA127VS
Lot no.	Lot specific
Agent	Mycoplasma pneumoniae - Strain FH
Culture system	Broth culture
Preparation	Harvested organisms are washed and processed in a two step detergent extraction process. Antigen is enriched for P1 adhesin protein and is presented in phosphate solubilising buffer containing 2% n-octylglucopyranoside.
Inactivation	By detergent treatment. Inactivity is confirmed by attempted growth under original culture conditions. Since no procedure can guarantee absolute sterility, the reagent should be handled with appropriate safety precautions.
Preservation	None
Protein concentration	Lot specific, specified in mg/mL. Determined by Bradford assay (Biorad) using BSA as standard.
Analysis	By SDS-PAGE and ELISA
Storage	-20°C to -80°C. Avoid repeated freeze-thaw cycles.
Date of manufacture	Lot specific
Expiry date	10 years from manufacture

Manufacturer:
Institut Virion\Serion GmbH
Friedrich-Bergius-Ring 19
D-97076 Würzburg
Germany
Tel. +49 931 3045 0
Fax +49 931 3045 100
E-Mail: dialog@virion-serion.de
Internet: www.virion-serion.de

FB-IV-062 A